





#1 CHOICE OF HOSPITALS AND MOTHERS

Choosing the right size PersonalFit™ breastshield:


1. Determine the size you're currently using. If you're not sure, look for the size embossed on your breastshield (see picture). 24mm (M) breastshields are provided with Medela breastpumps.

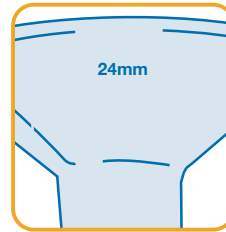


2. While pumping, compare your fit to the images below as a sizing guideline.

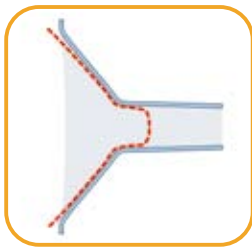

Correct fit

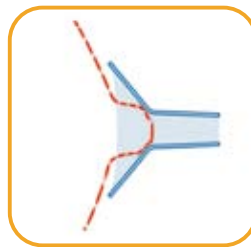

Your breastshield is **too small**;
try a larger size

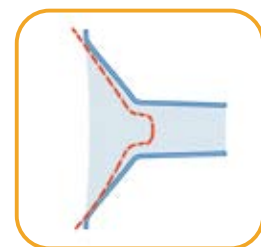

Your breastshield is **too large**;
try a smaller size



Only Medela offers PersonalFit™ breastshields to help ensure comfort and efficient pumping.



21mm



24mm



27mm



30mm



36mm

(provided with
Medela breastpumps)

QUESTIONS?

Call Medela Customer Service at 1-800-435-8316