Double Select™

BREASTPUMP

BOMBA DE LACTANCIA

Manufactured exclusively by and for Medela, Inc.
Fabricado exclusivamente por y para Medela, Inc.
P.O. Box 660, McHenry, IL 60051-0660
1101 Corporate Drive, McHenry, IL 60050 USA
Phone/Teléfono: 800-435-8316, 815-363-1166
Fax: 815-363-1246

Email/Correo electrónico: customer.service@medela.com

www.medela.com

**Instructions &
Assembly**

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Printed in the USA. Medela, Lactina and Symphony are registered trademark of Medela, Inc. Double Select, Pump & Save, Quick Clean and Micro-Steam are trademarks of Medela, Inc. in the USA and in other countries.

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English

Table of Contents

Important Safeguards	1
Uses	2
Health & Safety Information	2
Hygiene, Cleaning, Storing Breastmilk	
Assembly Instructions	6
Battery & Electric Operation, To Begin Pumping	
Pumping Tips	9
Accessories	9
Troubleshooting	10
Warranty	12
Ordering Parts	13
Breastpump Parts List	13
Parts	28
Español - Instrucciones	14

IMPORTANT

This product has been designed for single user use and should not be shared between mothers.

Disassemble all parts prior to cleaning and sterilization. Especially be sure to remove the valve from the breastshield and separate the white membrane from the valve prior to cleaning or sterilization. See parts 1, 2 and 3 on breastpump parts lists (Page 28).

IMPORTANT SAFEGUARDS

This is a single-user pump, which cannot be autoclaved or sterilized without destroying the product. Use by more than one person may present a health risk and voids the warranty.

When using electrical devices, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USE

DANGER: To reduce the risk of electrocution:

1. Always unplug electrical devices immediately after using.
2. Do not use while bathing.
3. Do not place in or drop into water or other liquid.
4. Do not reach for any electrical device if it has fallen into water. Unplug immediately.

WARNING: To reduce the risk of burns, electrocution, fire or injury to persons:

1. The pump should never be left unattended when plugged into an electric outlet.
2. Use the pump only for its intended use as described in this manual.
3. The use of accessory attachments not recommended by the manufacturer may cause hazards.
4. Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Keep the cord away from heated surfaces.
6. Do not use outdoors.
7. Plug the transformer into Double Select™ first and then into wall socket.
8. Only use the transformer that comes with the Double Select™.
9. Do not use the Vehicle Lighter Adapter with a Double Select™ Breastpump.
10. This product is a personal care item and should not be shared between mothers.

SAVE THESE INSTRUCTIONS

Uses

- Collect breastmilk during short-term absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement or sore nipples. Pumping with simultaneous massage is helpful to relieve engorgement.

NOTE: For long-term pumping when the baby is not available to feed at the breast at least part of the time every day (a premature baby, for example), use of a full-size electric pump (Lactina® or Symphony® Breastpump) is suggested. These pumps can be rented inexpensively for home use. For the names of rental stations or breastfeeding specialists in your area, call 1-800-TELL YOU (1-800-835-5968).

Cleaning & Sterilization

Follow these cleaning instructions in the hospital or at home unless you are otherwise instructed by your healthcare professional.

Hygiene

Wash hands before touching breasts or containers, and avoid touching the inside of containers or lids.

Preparation

Prior to cleaning, take apart all parts of your breastpump accessory kit. Be especially sure to remove the valve from the breastshield and separate the membrane from the valve prior to cleaning.

Prior To First Use

To Sterilize in the Hospital:

All disassembled kit parts (parts 1-6, page 28) can be autoclaved to a maximum of 272° F for 3 minutes at 29 PSIG or 250° F for 15-20 minutes at 15 PSIG.

To avoid damage during and immediately after sterilization:

- Sterilize parts without applying external pressure to them.
- Allow parts to cool before applying external pressure.

NOTE: A white or gray tint may appear on tubing and other parts after sterilization. This is normal and will not affect the breastpump's performance.

If gas sterilization is used, residual testing is recommended to determine the proper length of aeration.

To Sanitize at Home:

Disassemble all parts that come in contact with breast or milk, and sanitize by boiling for 10 minutes.

After each use:

- Disassemble and wash all parts that come in contact with the breast and milk in warm soapy water.
- Rinse in clear water.
- Air-dry on clean towel and cover parts when not in use.

All parts may also be washed in top rack of dishwasher.

For Quicker Cleaning:

Medela offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection.

Tubing Care:

Inspect tubing after each pumping session for condensation and or milk.

If condensation appears in tubing:

To eliminate condensation after you have completed pumping, continue running the pump with the tubing attached for another 1-2 minutes or until dry.

If milk appears in tubing:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing, pump connector, piston, cylinder, membranes and membrane caps (parts vary by pump used). Wash in soapy water, then rinse in clear cold water.
3. Shake water droplets out of tubing and hang to air-dry. For faster drying, attach tubing to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubing to dry.

Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Breastmilk Storage (For Healthy Term Babies)					
	Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Self-contained Refrigerator Freezer Unit	Deep Freezer
Freshly expressed breastmilk	4 hours at 66-72°F (19-22°C) ¹	24 hours at 59°F (15°C) ¹	5 -7 days at 32-39°F (0°-4°C) ²	3-4 months ³	6-12 months at 0°F (-19°C) ³
Thawed breastmilk (previously frozen)	Do not store	Do not store	24 hours ⁴	Never refreeze thawed milk	Never refreeze thawed milk

¹ Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: Pediatrics, Vol. 97, No. 4, April 1996. pp 492-497. (4 hours at 77° F/25° C).

² Sosa, Roberto; Barness, Lewis: AJDC, Vol. 141, Jan. 1987.

³ Lawrence R, and Lawrence R: Breastfeeding: A Guide For the Medical Profession, 1999, p.894.

⁴ Arnold L; Recommendations for Collection, Storage and Handling of a Mother's Milk for Her Own Infant in the Hospital Setting, 3rd Edition. The Human Milk Banking Association of North America, Inc. 1999, p. 18.

IMPORTANT NOTE: *If the temperature in the room, car or outdoors exceeds 77° F / 25° C, chill milk immediately to preserve freshness.*


Assembly Instructions

1 Follow cleaning instructions on pages 2-4.

2 Snap white membranes onto yellow valves until membrane lies completely flat. (Prior to cleaning, separate these two pieces.)

3 Push assembled valves and membranes onto breastshields.

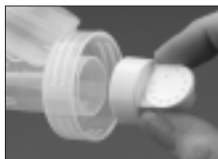
4 Screw containers onto breastshields.

5 Insert white ends of tubing into openings in breastshields.

6 Connect the other end of the tubing to either side of the motor unit. If single pumping (one side), be sure the cap is securely closed on the second side.

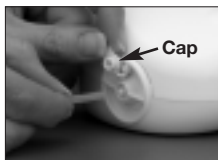Battery Operation

The Double Select™ Breastpump can be powered by two (2) C 1.5V alkaline batteries.

1 To install the batteries, press the tab (PRESS) down to open the cover of the battery compartment.

Insert the batteries in the correct position (+ and -) as indicated inside the battery compartment and snap the cover back into place.

CAUTION: When using the pump with batteries, the pumping cycle may slow as the batteries wear down. Slower cycles may cause some discomfort because the vacuum will be held longer on the breast. If this happens, reduce the vacuum with the adjustment regulator and replace the batteries with fresh ones as soon as possible.

NOTE: The batteries do not need to be removed to use electricity to power the pump.

Electric Operation

1 Plug the metal fitting on the end of the transformer cord into the hole on the back of the motor unit. (Use only the Medela transformer #9200013)

To Begin Pumping

1 Adjust the vacuum to minimum (down) to begin pumping. Increase the vacuum during pumping as comfort allows.

2 Press the ON ("I") switch to begin pumping.

3 Center the breastshields firmly over the nipples and start pumping.

IMPORTANT: When using the pump, do not overfill or tilt the containers.

Pumping Tips

It is important that you be as comfortable and relaxed as possible when pumping. Learning relaxation techniques may be helpful. Many mothers find that focusing on a photograph of their baby is useful.

When double pumping, it may take about 10-15 minutes to collect milk from both breasts. Single pumping may take 10-15 minutes per side.

The following steps of caution should be followed to prevent overflowing milk into the motor:

- Care should be taken not to over fill the container(s).
- Do not tilt or lay down the container when milk is in it and the tubing is attached.
- Remove the tubing from the breastshield before laying down the breastshield.

Accessories

Spare Membranes

Extra membranes for all Medela Breastpumps.
6 membranes per pack.

#87088 (US)

Quick Clean™ Micro-Steam Bags

Requires just water and a microwave—eliminates 99.9% of all harmful bacteria and germs from most breastpump parts and feeding accessories. 5 Micro-Steam bags per box.

#87024 (US)

Troubleshooting


If milk accidentally gets into the tubing/motor, immediately follow these steps:

- 1 Remove tubing from the motor. Wipe milk off the motor with a paper towel or tissue. Run motor for at least 30 seconds to clear out any milk that remains inside.


- 2 Take special care to clean the vacuum ports in the front sides of the motor.

Start by removing the covers; turning in the direction of the arrow (counter clockwise) and pull out.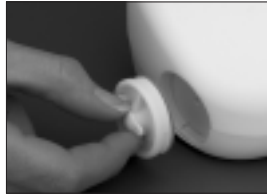

- 3 Remove the gasket(s). Wash gasket(s), tubing* and cover(s) with warm soapy water. Rinse in clear water and dry thoroughly.

NOTE: Only the side with potential overflow needs to be cleaned.

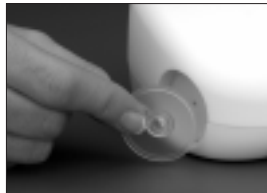- 4 Wipe the area on the motor unit under the gasket and on the bottom around the vent hole.

Replace the dried gasket(s), tubing* and cover(s).

Put the tubing back on the breast shield and resume pumping. When pumping is completed, repeat these steps.

If There Is No Vacuum

If you do not feel any vacuum at the breast or if the pumping action is too weak:

- Check that the breastshield forms a complete seal around the breast.
- Check to see that the membrane is securely attached to the valve on the underside of the breastshield. Ensure that membrane is clean and that the flap is moveable and not dried onto the valve head.
- For single pumping, check whether the second connection on the pump has been closed.
- Check that the gaskets are clean and dry.
- Check with fresh batteries or electrical if not powering unit with batteries.
- Check that the connector tube is firmly plugged into the breastshield.

* See page 4, “#3 Air-Dry” for further information on drying tubing.

90-Day Limited Warranty

This product is warranted by Medela, Inc., to the original retail purchaser to be free from defects in material and workmanship for 90 days (six months for pump mechanism) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

You may call 1-800-435-8316 for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela, prepaid, together with your dated bill of sale or other proof of purchase, a brief statement of the problem and your return authorization number to the following address:

**Attn: Medela, Inc.
P.O. Box 660
McHenry, IL 60051-0660 USA**

Ordering Parts

Many Medela locations have parts available for sale. Call the location where you purchased the breastpump or call 1-800-TELL YOU in USA or Canada for a location in your area, or call 1-800-435-8316 or 1-815-363-1166 extension 114 and order with VISA, MasterCard, American Express, or Discover Card. Prices subject to change without notice.

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