# Pump In Style<sup>®</sup> Advanced & Advanced Backpack

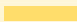
BREASTPUMP  
BOMBA DE LACTANCIA  
TIRE-LAIT

[www.medela.com](http://www.medela.com)

Medela, Inc.,  
P.O. Box 660, McHenry, IL  
60051-0660, USA  
Ph/Tel./Tél.: 1-800-435-8316  
Fax/Télé.: 1-815-363-1246  
Email/Correo electrónico/Courriel:  
[customer.service@medela.com](mailto:customer.service@medela.com)

Medela Canada, Inc.,  
4090B, Sladeview Crescent,  
Unit 2, Mississauga, Ontario,  
Canada L5L 5Y5  
Ph/Tel./Tél.: 1-905-608-7272  
Fax/Télé.: 1-905-608-8720  
Email/Correo electrónico/Courriel:  
[info@medela.ca](mailto:info@medela.ca)

-  **Instructions for use**
-  **Instrucciones de uso**
-  **Mode d'emploi**

Printed in the USA. Medela, 2-Phase Expression, Lactina and Pump In Style are registered trademarks of Medela in the USA and a trademark in other countries. CSF, DayCarrier, PureLan, Quick Clean and PersonalFit are trademarks of Medela, Inc. in the USA and in other countries. Applications for registration of Medela is pending in other countries. US Patent #'s 4,857,051; 4,929,229; 5,776,098. Additional patent pending. All rights reserved.

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## IMPORTANT SAFEGUARDS

*This is a single user product. Use by more than one person may present a health risk and performance questions, and voids the warranty.*

*When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:*

### READ ALL INSTRUCTIONS

**DANGER:** To reduce the risk of electrocution:

1. Always unplug electrical products immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product if it has fallen into water. Unplug immediately.

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged into an electric outlet.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Keep the cord away from heated surfaces.
6. Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
7. Plug the transformer into Pump In Style Advanced product first and then into wall socket.
8. Only use the transformer that comes with the Pump In Style Advanced Breastpump.
9. Only use the Medela Pump In Style Advanced Battery Pak with the Medela Pump In Style Advanced Breastpumps.
10. This product is a personal care item and should not be shared between mothers.
11. Make sure the voltage of the transformer is compatible with the power source.
12. Pump only with the motor in an upright position. (See page 50)
13. Never use while sleeping or when drowsy.

**SAVE THESE INSTRUCTIONS**

## Table of Contents-English

Uses.....	1
Health & Safety Information .....	1-4
Hygiene, Cleaning, Storing Breastmilk .....	
Troubleshooting .....	5
Assembly Instructions .....	
Pump In Style Advanced Breastpump.....	6-9
Manual Pumping.....	10
Pump In Style Advanced Battery Pak .....	11
Features of the Pump In Style Advanced Breastpump.....	12
Warranty .....	13
Ordering Parts .....	14
Breastpump Parts List.....	14
Breastfeeding Accessories (sold separately) .....	15
Illustration of Parts.....	50

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## Uses

- Collect breastmilk at work or during other absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement with single pumping. Single pumping with simultaneous massage is helpful to relieve engorgement.

## Health & Safety Information

**Important:** This product is a single user item. Use by more than one user may pose a health risk and performance questions and voids the warranty.

### Hygiene

Wash hands before touching breasts or containers and avoid touching the inside of containers or lids.

### Cleaning

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Prior to cleaning, take apart all parts of your breastpump accessory kit. Especially be sure to remove the yellow valve from the breastshield and separate the white membrane from the valve prior to cleaning. See parts 6 - 8 on the illustration (Page 50).

### Prior to first use:

Disassemble all parts of your kit and breastpump that come in contact with the breast and milk (except pump and bag or backpack). Sanitize by boiling for 10 minutes.

### After each use:

Disassemble and wash all parts that come in contact with the breast and milk in warm, soapy water (except pump and bags).

- Rinse in clear water.

- Air-dry on clean towel and cover parts when not in use.
- Use a damp (not wet) cloth to wipe the bag and diaphragm cap of breastpump after each use (see part 14 on parts illustration Page 50).

**For Quicker Cleaning:**

Medela offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection.

**Tubing Care:**

Inspect tubing after each pumping session for condensation and/or milk.

**If condensation appears in tubing:**

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry.

**If There Is An Overflow:**

*If milk accidentally gets into the tubings:*

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing. Wash in soapy water, then rinse in cold clear water.
3. Shake water droplets out and hang to air-dry. For faster drying, attach tubings to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubings to dry.

*If milk accidentally gets behind the diaphragm cap:*

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing.
3. Snap off the diaphragm cap using thumb tab.
4. Be sure the yellow plug is not plugged into any ports prior to cleaning. Wash the diaphragm cap in soapy water, then rinse in cold, clear water.

5. Use a damp (not wet) cloth to wipe the diaphragm and bag. Do not immerse the motor unit or any of the bags in water.
6. When components are completely dry, reassemble breastpump and kit.

**Storing Breastmilk**

*Check with your hospital for specific storage instructions.*

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Breastmilk Storage (For Healthy Term Babies)					
	Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Self-contained Refrigerator Freezer Unit	Deep Freezer
Freshly expressed breastmilk	4 hours at 66-72°F (19-22°C) <sup>1</sup>	24 hours at 59°F (15°C) <sup>1</sup>	5 - 7 days at 32-39°F (0°C) <sup>2</sup>	3-4 months <sup>3</sup>	6-12 months at 0°F (-19°C) <sup>3</sup>
Thawed breastmilk (previously frozen)	Do not store	Do not store	24 hours <sup>4</sup>	Never refreeze thawed milk	Never refreeze thawed milk

<sup>1</sup> Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: Pediatrics, vol. 97, No. 4, April 1996, pp 492-497. (4 hours at 77° F/25° C).

<sup>2</sup> Sosa, Roberto; Barnes, Lewis: AJDC, Vol. 141, Jan. 1987.

<sup>3</sup> Lawrence R, and Lawrence R: Breastfeeding: A Guide For the Medical Profession, 1999, p.894.

<sup>4</sup> Arnold L: Recommendations for Collection, Storage and Handling of a Mother's Milk for Her Own Infant in the Hospital Setting, 3rd Edition. The Human Milk Banking Association of North America, Inc. 1999, p. 18.

**Important Note:** *If the temperature in the room, car or outdoors exceeds 77° F / 25° C, chill milk immediately to preserve freshness.*

---

### Storing Breastmilk in the Pump In Style Advanced or Pump In Style Advanced Backpack Bag:

- Pump In Style Advanced and Pump In Style Advanced Backpack require one contoured cooling element (#8117010—included) for cooling up to four containers of breastmilk in the included DayCarrier™ bag.

As soon as breast pumping is finished, place the milk container or bag in the compartment with the cooling elements. If the cooling element(s) are frozen solid, they will safely cool your breastmilk for up to 12 hours. When you return home transfer the containers to the refrigerator or freezer.

### Defrosting

- Thaw milk overnight in the refrigerator, or hold the bottle under warm running water to quickly thaw. You can also place the sealed container in a bowl of warm water for 20 minutes to bring it to body temperature.
- Thawed milk is safe in the refrigerator for 24 hours. **Do not refreeze.**

### Caution:

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Never microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breastmilk.
- If adding expressed breastmilk to container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

---

## Troubleshooting

### If There Is Low or No Suction

1. Make sure all connections of the kit and breastpump are secure.
2. If single pumping, make sure the port plug is placed into one of the single pumping ports.
3. Make sure the white membrane is laying flat against the yellow valve head. Check that the yellow valve head is clean and not damaged. Refer to the assembly instructions.
4. Make sure the diaphragm cap or faceplate on the front of the breastpump fits evenly and securely.
5. Make sure the ends of the tubings are fit securely into the backs of the breastshields and into the ports on the diaphragm cap or faceplate.

### Other concerns

6. Take care not to kink tubing(s). If milk is seen backing up into tubing, check that tubing is not kinked or pinched.
7. When traveling outside the U.S., use battery power only.
8. If you experience discomfort while pumping contact a breastfeeding specialist. You may be more comfortable with a PersonalFit™ or SoftFit™ Breastshield. Call 1-800-TELL-YOU to locate a breastfeeding specialist in your area.
9. In the US, call Medela at 800-435-8316 or 815-363-1166. In Canada, call Medela at 800-435-8316 or 905-608-7272. In other countries, contact your local distributor or the location where you purchased your pump.

## Assembly Instructions - Pump In Style Advanced Breastpumps

- 1 Make sure breastshield and connector are securely attached.

- 2 Snap membranes onto valves until membranes lie completely flat.

\*Be sure to separate membranes & valve heads when cleaning and completely dry prior to assembly.

- 3 Push assembled valves & membranes onto breastshield connectors.

- 4 Screw containers into breastshield connectors.

- 5 Insert hard yellow adapter ends of tubing into openings in breastshield connectors.

- 6 Insert other ends of tubing into ports on diaphragm cap. To remove tubing, pull straight out of tubing ports; do not wiggle tubing or pull at an angle.

- 7 **If single pumping:** insert port cap into the unused port.

- 8 **If double pumping:** push port cap onto post.

- 9 Plug the metal fitting on end of transformer or Battery Pak cord into power jack on breastpump. Plug the transformer into a standard electrical outlet.

10 Center breastshields over your nipples.

**IMPORTANT:** Do not tilt or overfill containers when pumping.

**IMPORTANT**

Take care not to kink tubing(s) while pumping.

11 Switch on Pump In Style Advanced and begin Stimulation Phase. Adjust speed by turning the Speed Select Switch clockwise to a cycling speed most comfortable for you.

12 After two minutes, the Stimulation Phase will automatically change to the Expression Phase. If let-down/milk flow is achieved prior to automatic switchover in two minutes, gently press the let-down button indicated by  and the pump will immediately change to the Expression Phase.

**Tip:** Once you are finished pumping; unplug tubing from back of breastshield prior to setting down bottles.

**Removing and securing the diaphragm cap:**

Only remove diaphragm cap when cleaning is necessary. See pages 1-3 for detailed cleaning information.

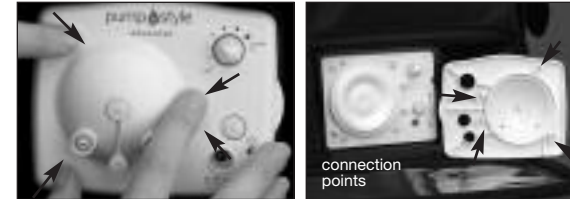
**To remove the diaphragm cap:**

1. Turn off pump, and unplug from power source.
2. Remove tubing (see instructions on page 7).
3. Snap off the diaphragm cap using the thumb tab. (see left photo)

**To secure the diaphragm cap:**

Hold back of pump with one hand while pushing on the diaphragm cap until it snaps into place.

**NOTE:** The diaphragm cap must be completely snapped together at all connection points (see arrows) to ensure adequate vacuum and optimum pump performance.

## Assembly Instructions for Manual Pumping

Note: Follow steps 1-4 on page 6.

- 5 Slip rubber seal onto piston until it snaps into place—the larger side faces the handle. To ensure proper seal, hold rubber seal firmly and twist piston.

- 6 Slide piston into cylinder.

- 7 Screw cylinder into breastshield assembly.

- 8 Set vacuum regulator ring to “MIN.” Adjust for comfort once pumping begins.

- 9 Center breastshield over your nipple, and begin pumping.

### IMPORTANT

Do not overfill or tilt the containers when pumping.

## Pump In Style Advanced Battery Pak Instructions

### The Pump In Style Advanced Battery Pak

The Pump In Style Advanced Battery Pak is intended for use with the Pump In Style Advanced Breastpump ONLY. The Battery Pak uses 10 AA alkaline batteries, or, for more consistent pump performance, use rechargeable batteries. Each set of batteries will provide approximately 2 hours of pumping time. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.

### Battery Insertion

Slide off battery covers. Place the NEGATIVE (-) end of the battery against the front of the spring. Push the battery in and down in one motion until the battery snaps into place. The POSITIVE (+) end of the battery should be near the positive sign on the Battery Pak. NOTE: If the battery is inserted in the opposite direction, the battery may not function correctly. Five batteries must be inserted on each side of the Pak. Replace battery covers.

### Battery Removal

Slide off battery covers. Insert a BLUNT, NON-METALLIC tool into the removal slot on either side of the Battery Pak. Gently pry the battery out. Replace battery covers.

The pouch fits inside the main bag or motor bag, along with the transformer. Or for easier access, the velcro strap allows for attachment to the pump shoulder strap.

## Features of the Pump In Style Advanced Breastpumps

### Removable motor unit

- Leave the motor unit, in its own discreet black bag, where you pump. Enjoy the extra room in your bag.

### Removable cooler carrier

- Carry the cooler carrier alone on those days when you don't need a big bag.

### PersonalFit™ Breastshield System

- Exclusive Medela PersonalFit Breastshield System for fit and comfort options.

### Extra storage pockets

- The Pump In Style Advanced Breastpump has outside pockets on each end and across the full width of one side. There is also a mesh pocket on the inside.
- The Pump in Style Advanced Backpack features extra compartments for water bottle, keys, pens, cell phone and credit cards.

### Multi-functional straps and handles

- The microfiber shoulder bag features a comfortable shoulder strap and stylish design.
- The microfiber backpack features ergonomic padded straps for hands free convenience.

### Optional DayCarrier™ Kit

- The DayCarrier Kit was designed to give you more freedom. Using the same cooler carrier that fits the Pump In Style Advanced Breastpump, it contains everything you need to leave with your baby at daycare. A second bag goes with you to safely store pumped milk.

## Medela Pump In Style Advanced Breastpumps

***This is a single user product. Use by more than one person may present a health risk and performance questions, and voids the warranty.***

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of 90 days (one year for pump mechanism) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE OR IN YOUR COUNTRY.

**In the US and Canada:** Before you make a claim under this warranty, or for additional information, call Medela Customer Service at 800-435-8316 (US & Canada), 815-363-1166 (US) or 905-608-7272 (Canada). You may also contact Medela at [customer.service@medela.com](mailto:customer.service@medela.com) (US) or [info@medela.ca](mailto:info@medela.ca) (Canada).

**In other countries:** Please contact your local distributor or the location where you purchased your pump; or you may write to Medela at the McHenry, IL, USA address.

**MEDELA, INC.**  
1101 Corporate Drive  
McHENRY, ILLINOIS, 60050  
USA  
[customer.service@medela.com](mailto:customer.service@medela.com)  
[www.medela.com](http://www.medela.com)

**MEDELA CANADA INC.**  
4090B Sladeview Crescent, Unit #2  
MISSISSAUGA, ONTARIO, L5L 5Y5  
CANADA  
[info@medela.ca](mailto:info@medela.ca)  
[www.medela.ca](http://www.medela.ca)

## Ordering Parts

If your retailer doesn't carry the parts you need, in US and Canada, you can call 800-435-8316 (US and Canada), 815-363-1166 (US) or 905-608-8720 (Canada) and order with VISA, MasterCard, American Express, or Discover Card. In other countries please contact your local distributor. See our web-site, [www.medela.com](http://www.medela.com), for a distributor listing.

## Parts (See illustration Page 50)

Item	Order Number
1. Form Fit Cooling Element .....	#8117010
2. Tubing .....	#8007212
3. Lactina® Piston/Cylinder Assembly .....	#8007190
4. Solid Container Lid .....	#8107057
5. 2-Piece Container Lid: Disc .....	#8100473
Collar .....	#8107062
6a. PersonalFit Breastshield .....	#87073
6b. PersonalFit Connector .....	#87071
7&8. Valves with Membranes .....	#87089
8. Membrane .....	#87088
9. 5 oz. (150 ml) container .....	#8117009
10. Container Stand .....	#8100462
11. Transformer* .....	#9207041
12. Battery Pak* .....	#9017006
13. Diaphragm Cap Assembly** .....	#6007094

### Accessories (sold separately)

DayCarrier Kit .....	#57016
PowerPak™ (Not shown) .....	#6703201
CSF™ Bags, Economy Pak .....	#87010
CSF™ Bags, Starter Pak .....	#87013
PureLan™ 100, 1/4 oz tube (7gr) .....	#87021
PureLan 100, 1.3 oz tube (36.8gr) .....	#87022B
Quick Clean Micro-Steam Bags .....	#87024
Vehicle Lighter Adapter .....	#67153

\* For use with Pump In Style Advanced only.

\*\* For use with Pump In Style Advanced units sold after June, 2006 only.

## Breastfeeding Accessories

(sold separately)

Please check with your local retailer for any of these Medela accessories to help make your breastfeeding experience more convenient, comfortable and enjoyable.

### DayCarrier Kit

#57016

Insulated, portable cooler carrier complete with 4 milk containers, 4 solid lids, 4 two-piece lids, 4 orthodontic slow-flow nipples, 2 caps, and a cooling element. Zippered, easy clean micro-fiber exterior.

### Pump & Save™ Breastmilk Bags

#87033 20-pack, #87034 50-pack

Designed especially for breastmilk storage, Pump & Save Bags offer unsurpassed protection during refrigeration or freezing. Leak-proof, easy-to-close zipper top plus an exclusive self-stick strap to attach directly to your pump.

### PureLan 100

1.3 oz. tube #87022B, .25 oz. tube #87021

Ultra pure, hypoallergenic, non-toxic form of USP Modified Lanolin soothes and helps heal sore nipples.

### Quick Clean Micro-Steam Bags

#87024

Requires just water and a microwave — eliminates 99.9% of harmful bacteria and germs from most breastpump parts and feeding accessories. 5 Micro-Steam bags per box.

### Disposable Bra Pads

#89973

Prevent leakage and keep breasts dry and comfortable with unique moisture absorbing materials. Smooth and soft next to tender skin. Contouring pleats ensure a discreet, feminine line under clothing. Self-adhesive tab keeps bra pad in place. Contains 30 pads.

For a breastfeeding specialist or to locate Medela rental or retail locations, call the Breastfeeding National Network™ (BNN™) at 1-800-TELL YOU; or visit the Medela website: [www.medela.com](http://www.medela.com).

The Medela Customer Service Team can be reached at 1-800-435-8316.

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