Cleaning Instructions – Bottles & Lids

1. Prior to first use, bring water to boil then place lids & bottles in boiling water for 10 minutes.
2. For regular cleaning, all components may be washed on top rack of dishwasher. Labeling lids and breastmilk bottles may be sanitized in boiling water for 10 minutes. To hand wash, rinse in cool water to remove breastmilk, wash in warm soapy water and rinse with clear water. Air dry in clean area and store when not in use. Do NOT store wet or damp parts. Labeling lids and breastmilk bottles may also be sanitized using a Quick Clean™ Micro-Steam™ bag.

Instructions for Use Organizer Tray

1. Place newest bottles or bags of breastmilk in the tray just behind the “IN” arrow, naturally rotating bottles or bags already in the tray toward the “OUT” arrow.
2. When ready to feed, remove bottle or bag from the side of the tray with the “OUT” arrow.
3. Arrows remind the person feeding baby that older breastmilk is used first to help prevent wasted/spoiled milk.

Instructions for Use Breastmilk Labeling Lids

Simple and secure!

Just lock into place for effortless, reliable tracking.

1. Screw the lid onto the breastmilk bottle and turn gently until secure.
2. Grip the outer section of the lid and pull up to unlock.
3. Turn the outer section of lid to line up the day of the week with the a.m./p.m. markings on the outer ring. Middle hash mark can be used to indicate afternoon.
4. Push outside of lid down to click and lock into place. Even if the lid is bumped or the bottle is dropped, the settings will remain unchanged.

IMPORTANT: Plastic bottles and component parts become brittle when frozen and may break when dropped. Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over. Take appropriate care in handling bottles and components. Do not use the breastmilk if bottles or components become damaged.

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