## Step 1: Choose your program

Use **INITIATE** immediately after birth:

- Within the first 5 days, **OR**;
- Until you've pumped at least 20 mL in each of your last 3 sessions

Use **MAINTAIN** after milk comes in:

- Once you've pumped at least 20 mL in each of your last 3 sessions, **OR**;
- Beginning day 6, whichever comes first

## Step 2: Select program and get started

Press  to start the breastpump.

### INITIATE program

To select the **INITIATE** program, press  while the screen displays:

After selecting the **INITIATE** program, the screen will display:

The **INITIATE** program alternates between the stimulation and expression phase, with several pauses and turns off automatically when the session is completed. Complete the entire program.

### MAINTAIN program

To select the **MAINTAIN** program, wait **5-10 seconds**, while the screen displays:

The **MAINTAIN** program will start automatically. The screen will then display:

While pumping in the **MAINTAIN** program, the stimulation phase will last 2 minutes. If your milk starts flowing before the expression phase begins, press  to begin the expression phase.

To achieve Maximum Comfort Vacuum<sup>™</sup>, adjust knob to optimum suction level.

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