Pump In Style[®] Original

BREASTPUMP
BOMBA DE LACTANCIA

www.medela.com

Medela, Inc.,
P.O. Box 660, McHenry, IL
60051-0660, USA
Ph/Tel: 1-800-435-8316
Fax: 1-815-363-1246
Email/Correo electrónico:
customer.service@medela.com

Instructions for Use

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Printed in the USA. Medela and Pump In Style are registered trademarks of Medela in the USA and a trademark in other countries. CSF, DayCarrier, Micro-Steam, PersonalFit, PureLan, Quick Clean and SoftFit are trademarks of Medela, Inc. in the USA and in other countries. Applications for registration of Medela is pending in other countries. US Patent #'s 4,857,051; 4,929,229; 5,776,098. Additional patent pending. All rights reserved.

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IMPORTANT SAFEGUARDS

This is a single user product. Use by more than one person may present a health risk and performance questions, and voids the warranty.

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS

DANGER: To reduce the risk of electrocution:

1. Always unplug electrical products immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product if it has fallen into water. Unplug immediately.

WARNING: To reduce the risk of burns, electrocution, fire or injury to persons:

1. This product is a personal care item and should not be shared among mothers.
2. A product should never be left unattended when plugged into an electric outlet.
3. Close supervision is necessary when this product is used by, on, or near children or invalids.
4. Use the product only for its intended use as described in this manual.
Do not use attachments not recommended by the manufacturer.
5. Never operate this product if it has a damaged cord or plug, if it not working properly, if it has been dropped or damaged, or dropped into water.
6. Keep the cord away from heated surfaces.
7. Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.
8. Plug the transformer into Pump In Style product first and then into wall socket.
9. Only use the transformer that comes with the Pump In Style Original Breastpump (#9207010).
10. Make sure the voltage of the transformer is compatible with the power source.
11. Pump only with the motor in an upright position (see page 30).
12. Never use while sleeping or when drowsy.

SAVE THESE INSTRUCTIONS

Table of Contents - English

Uses.....	1
Health & Safety Information.....	1-4
Hygiene, Cleaning, Storing Breastmilk	
Troubleshooting.....	5
Assembly Instructions	
Pump In Style Original Breastpump.....	6-7
Manual Pumping.....	8
Pump In Style Original Battery Pak.....	9
Quick Tips From Other Pumping Moms.....	10
Breastfeeding Accessories.....	11
Warranty.....	12
Ordering Parts.....	13
Breastpump Parts List.....	13
Illustration of Parts.....	30
Índice - Español.....	14

Uses

- Collect breastmilk at work or during other absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement with single pumping. Single pumping with simultaneous massage is helpful to relieve engorgement.

Health & Safety Information

Important: This product is a single user item. Use by more than one user may pose a health risk and performance questions and voids the warranty.

Hygiene

Wash hands before touching breasts or containers and avoid touching the inside of containers or lids.

Cleaning

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Prior to cleaning, take apart all parts of your breastpump accessory kit. Especially be sure to remove the yellow valve from the breastshield and separate the white membrane from the valve prior to cleaning. See parts 3 & 4 on the illustration (Page 30).

Prior to first use:

Disassemble all parts of your kit and breastpump that come in contact with the breast and milk (except pump and bag). Sanitize by boiling for 10 minutes.

After each use:

Disassemble and wash all parts that come in contact with the breast and milk in warm, soapy water (except pump and bag).

- Rinse in clear water.

- Air-dry on clean towel and cover parts when not in use.
- Use a damp (not wet) cloth to wipe the bag and diaphragm cap of breastpump after each use (see part 1 on parts illustration Page 30).

For Quicker Cleaning:

Medela® offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection.

Tubing Care:

Inspect tubing after each pumping session for condensation and/or milk.

If condensation appears in tubing:

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry.

If There Is An Overflow:

If milk accidentally gets into the tubings:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing. Wash in soapy water, then rinse in cold clear water.
3. Shake water droplets out and hang to air-dry. For faster drying, attach tubings to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubings to dry.

If milk accidentally gets behind the diaphragm cap:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing.
3. Snap off the diaphragm cap using thumb tab.
4. Be sure the yellow plug is not plugged into any ports prior to cleaning. Wash in soapy water, then rinse in cold, clear water.

5. Use a damp (not wet) cloth to wipe the diaphragm and bag. Do not immerse the motor unit or the bag in water.
6. When components are completely dry, reassemble breastpump and kit.

Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Breastmilk Storage (For Healthy Term Babies)					
	Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Self-contained Refrigerator Freezer Unit	Deep Freezer
Freshly expressed breastmilk	4 hours at 66-72°F (19-22°C) ¹	24 hours at 59°F (15°C) ¹	5 - 7 days at 32-39°F (0°C) ²	3-4 months ³	6-12 months at 0°F (-19°C) ³
Thawed breastmilk (previously frozen)	Do not store	Do not store	24 hours ⁴	Never refreeze thawed milk	Never refreeze thawed milk

¹ Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: Pediatrics, vol. 97, No. 4, April 1996. pp 492-497. (4 hours at 77° F/25° C).

² Sosa, Roberto; Barness, Lewis: AJDC, Vol. 141, Jan. 1987.

³ Lawrence R, and Lawrence R: Breastfeeding: A Guide For the Medical Profession, 1999, p.894.

⁴ Arnold L: Recommendations for Collection, Storage and Handling of a Mother's Milk for Her Own Infant in the Hospital Setting, 3rd Edition. The Human Milk Banking Association of North America, Inc. 1999, p. 18.

Important Note: *If the temperature in the room, car or outdoors exceeds 77° F / 25° C, chill milk immediately to preserve freshness.*

Storing Breastmilk in the Pump In Style Original Bag:

- Pump In Style Original requires one contoured cooling element (#8117010—included) for cooling up to four containers of breastmilk in the included DayCarrier™ bag.

As soon as breast pumping is finished, place the milk container or bag in the compartment with the cooling elements. If the cooling element(s) are frozen solid, they will safely cool your breastmilk for up to 12 hours. When you return home transfer the containers to the refrigerator or freezer.

Defrosting

- Thaw milk overnight in the refrigerator, or hold the bottle under warm running water to quickly thaw. You can also place the sealed container in a bowl of warm water for 20 minutes to bring it to body temperature.
- Thawed milk is safe in the refrigerator for 24 hours. **Do not refreeze.**

Caution:

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Never microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breastmilk.
- If adding expressed breastmilk to container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

Troubleshooting

If There Is Low or No Suction


1. Make sure all connections of the kit and breastpump are secure.
2. Make sure the port plug is placed into the unused single pumping port or onto the post.
3. Make sure white membrane is laying flat against the yellow valve head. Check that the yellow valve head is clean and not damaged. Refer to the assembly instructions.
4. Make sure the diaphragm cap or faceplate on the front of the breastpump fits evenly and securely.
5. Make sure the ends of the tubings are fitted securely into the backs of the breastshields and into the ports on the diaphragm cap or faceplate.

Other concerns

6. Take care not to kink tubing(s). If milk is seen backing up into tubing, check that tubing is not kinked or pinched.
7. If you experience discomfort while pumping contact a breastfeeding specialist. You may be more comfortable with a different size PersonalFit™ or a SoftFit™ Breastshield. Call 1-800-TELL-YOU to locate a breastfeeding specialist in your area.
8. In the US, call Medela at 800-435-8316 or 815-363-1166. In Canada, call Medela at 800-435-8316 or 905-608-7272. In other countries, contact your local distributor or the location where you purchased your pump.

Assembly Instructions - Pump In Style Original Breastpump

1. Make sure breastshield and connector are securely attached.

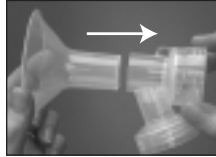
2. Snap membranes onto valves until membranes lie completely flat.

*Be sure to separate membranes & valve heads when cleaning and completely dry prior to assembly.

3. Push assembled valves & membranes onto breastshield connectors.

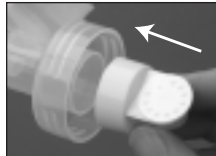4. Screw containers into breastshield connectors.

5. Insert hard adapter ends of tubing into openings in breastshield connectors.

6. Insert other ends of tubing onto ports on diaphragm cap. To remove tubing, pull straight out of tubing ports; do not wiggle tubing or pull at an angle.

7. **If single pumping:** insert port cap into the unused port.

8. **If double pumping:** push port cap onto center post.

9. Plug the metal fitting on end of transformer or Battery Pak cord into power jack on breast-pump. Plug the transformer into a standard electrical outlet.

10. Center breastshields over your nipples.

IMPORTANT: Do not tilt or overfill containers when pumping.

IMPORTANT

Take care not to kink tubing(s) while pumping.

11. Switch on Pump In Style Original and start pumping. Adjust speed and vacuum for your comfort.

Tip: Once you are finished pumping, unplug tubing from back of breastshield prior to setting down bottles.

Assembly Instructions for Manual Pumping with Pump In Style Original

Manual pump (#8007124) sold separately from Pump in Style Original Breastpump.

Note: Follow steps 1-4 on page 6.

- Slip rubber seal onto piston until it snaps into place—the larger side faces the handle. To ensure proper seal, hold rubber seal firmly and twist piston.

- Slide piston into cylinder.

- Screw cylinder into back of breastshield connector.

- Set vacuum regulator ring to “MIN.” Adjust for comfort once pumping begins.

- Center breastshield over your nipple, and begin pumping.

IMPORTANT

Do not overfill or tilt the containers when pumping.

Pump In Style Original Battery Pak Instructions

The Pump In Style Original Battery Pak is intended for use with the Pump In Style Original ONLY. The Battery Pak uses 8 AA alkaline batteries, or for more consistent pump performance, rechargeable batteries. Each set of batteries will provide approximately 3 hours of pumping time. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.

Battery Insertion

Slide off battery covers. Place the NEGATIVE (-) end of the battery against the front of the spring. Push the battery in and down in one motion until the battery snaps into place. The POSITIVE (+) end of the battery should be near the positive sign on the Battery Pak. NOTE: If the battery is inserted in the opposite direction, the battery may not function correctly. Four batteries must be inserted on each side of the Pak. Replace battery covers.

Battery Removal

Slide off battery covers. Insert a BLUNT, NON-METALLIC tool into the removal slot on either side of the Battery Pak. Gently pry the battery out. Replace battery covers.

Quick Tips From Other Pumping Moms:

- Assemble your breastshields and containers after cleaning so you're ready for your next pumping session. Two fully assembled breastshields and containers will fit in the bag.
- Purchase an extra cooling element, so you'll always have one frozen and ready-to-go.

- Remove the cooler carrier from the pump bag when you don't need to have the entire pump with you.


- Use the extra pockets in the Pump In Style Original bag to store and organize needed items, like bra pads, spare valves and membranes.

- The Pump In Style Original shoulder bag works great in places where there isn't a surface to place your pump, like a public washroom. Just hang the bag on a hook and drop down the bag's front panel to use as a stable surface for all your pump parts.

- The elastic loop on the front of the Pump In Style Original shoulder bag makes a great bottle holder while you're pumping. Use it to free-up one hand so you can assemble your breastshields or change out your breastmilk bottles. Be sure to remove the bottle from the loop before you pack-up your pump.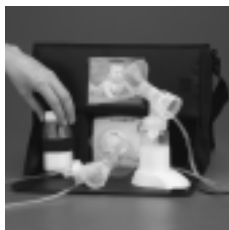

Breastfeeding Accessories

(sold separately)

Check with your local retailer for these Medela accessories to help make your breastfeeding experience more convenient, comfortable and enjoyable.

DayCarrier Kit #57016

Insulated, portable cooler carrier complete with 4 collection containers, 4 solid lids, 4 two-piece lids, 4 orthodontic slow-flow nipples, 2 caps and a cooling element. Zippered, easy-clean microfiber exterior.

PureLan™ 100 Lanolin #87022B

100% ultra-pure, lanolin soothes and rehydrates sore nipples. Also helps heal minor cuts, burns and abrasions. PureLan is safe for baby to ingest, so no need to remove before nursing.

Quick Clean Micro-Steam Bags #87024

Requires just water and a microwave – eliminates 99.9% of harmful bacteria and germs from most breastpump parts and feeding accessories. 5 Micro-Steam bags per box.

Disposable Nursing Bra pads #89974

Super absorbent materials prevent leakage and maintain dryness. Features contouring pleats to ensure a smooth, feminine shape, and adhesive tape backer to keep pad in place. Individually wrapped for on-the-go convenience.

Vehicle Lighter Adapter #67153

Frees you from the need for battery or electrical power. Runs off vehicle lighter.

Spare Membranes #87088

Extra membranes for all Medela Breastpumps
6 membranes per pack.

Medela Pump In Style Breastpumps

This is a single user product. Use by more than one person may present a health risk and performance questions and voids the warranty.

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of 90 days (one year for pump mechanism) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE OR IN YOUR COUNTRY.

In the US: Before you make a claim under this warranty, or for additional information, call Medela Customer Service at 800-435-8316, 815-363-1166. You may also contact Medela at customer.service@medela.com.

In other countries: Please contact your local distributor or the location where you purchased your pump; or you may write to Medela at the McHenry, IL, USA address.

**MEDELA, INC. P.O. Box 660
McHENRY, ILLINOIS, 60051-0660 USA**

customer.service@medela.com

www.medela.com

Ordering Parts

If your retailer does not carry the parts you need, you can call 800-435-8316 or 815-363-1166 and order with VISA, MasterCard, American Express, or Discover Card. In other countries than the USA, please contact your local distributor. See our website www.medela.com for a distributor listing.

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