# The PedalPump™

[www.medela.com](http://www.medela.com)

Medela, Inc., P.O.  
Box 660, McHenry, IL  
60051-0660, USA

Ph/Tél/Tel: 1-800-435-8316

Fax: 1-815-363-1246

Email/Courriel/Correo electrónico:  
[customer.service@medela.com](mailto:customer.service@medela.com)

Medela Canada, Inc.,  
4090B, Sladeview Crescent,  
Unit 2, Mississauga, Ontario,  
Canada L5L 5Y5

Ph/Tél/Tel.: 1-905-608-7272

Fax: 1-905-608-8720

Email/Courriel/Correo electrónico:  
[info@medela.ca](mailto:info@medela.ca)

Medela, Lactina, and Symphony are registered trademarks of Medela. Harmony, Natural Expression, SoftFit, QuickClean, PureLan, Classic, Breastfeeding National Network and BNN are trademarks of Medela.

Medela, Lactina y Symphony son marcas registradas de Medela. Harmony, Natural Expression, SoftFit, QuickClean, PureLan, Classic, Breastfeeding National Network y BNN son marcas registradas de Medela.

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Instructions & Assembly

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## Table of Contents

PedalPump™ Parts .....	2
Introduction to the PedalPump™ .....	3
PedalPump™ Assembly Instructions .....	3
PedalPump™ Instructions for Use .....	4

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
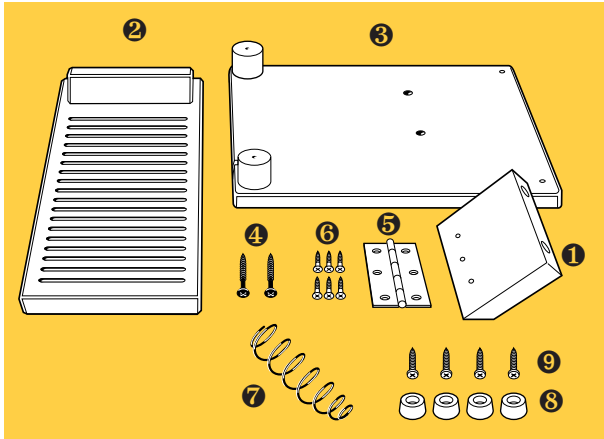
## Introduction to the PedalPump™

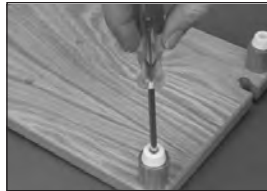The Medela PedalPump is uniquely powered by the strength of your leg and foot muscles. It allows for double and single pumping. The PedalPump is designed for use with any Medela Breastpump Accessory Kit that is compatible with the Lactina Breastpump.

### PedalPump™ Assembly Instructions:


- 1 Place the 4 rubber pads over holes in the corners of the base. Use pad screws to tighten rubber pads into base, just until screw heads touch rubber pads.

- 2 Use 3 of the hinge screws to tighten hinge into holes on pedal support. Use the 3 remaining hinge screws to tighten other side of hinge into pedal.

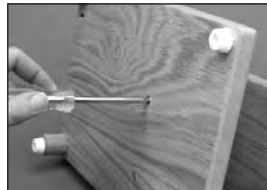- 3 Align pedal support over the 2 holes in the base. Use 2 base screws to tighten bottom of base to pedal support.


For detailed cleaning and assembly see the instructions that came with your kit. Assemble your breast pump accessory kit according to the following instructions.

### PedalPump™ Instructions for Use:

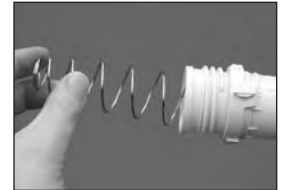- 1 Slide piston into cylinder.


- 2 Insert large end of spring into cylinder.

- 3 Screw cylinder tightly into pump connector.

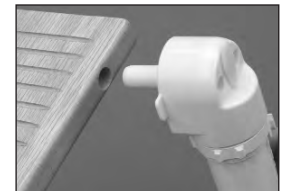- 4 Push pump connector into hole on side of pedal.

- 5 Insert piston into slot on base, and rotate until snug.

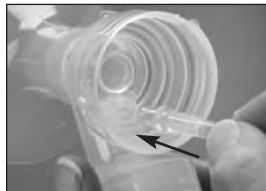
- 6 Push assembled valve & membrane onto breastshield assembly.

- 7 Screw container into breastshield assembly.

- 8 Insert clear ends of tubing into openings in connectors.

- 9 Insert other ends of tubing into ports on pump connector.

5

- 10 If single pumping, insert plug into unused port.

- 11 Set vacuum regulator to "MIN." Adjust for comfort once pumping begins.

- 12 Center breastshields over your nipples.

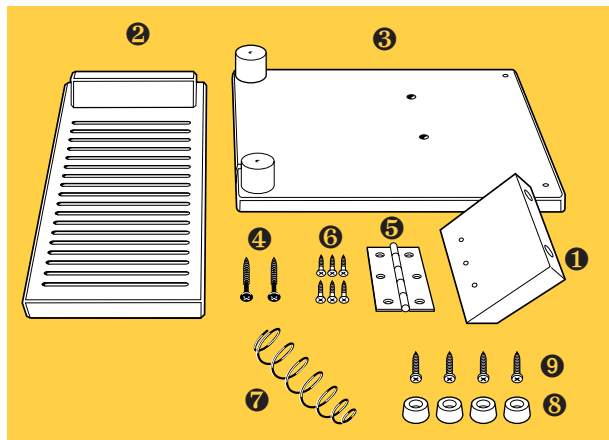
- 13 While sitting, position your foot on the pedal. Begin pumping by raising and lowering your foot on the pedal, pushing down with your heel as you raise your toes. Adjust foot position for comfort.

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