

# Harmony™ Assembly Instructions



1. Place silicone diaphragm/stem into pump handle and press firmly around entire edge to ensure proper seal.




2. Insert stem of handle assembly into opening on top of pump and push until it snaps in. Use of container stand (included) is optional. See enclosed instruction booklet for complete assembly instructions.

Upgradeable to electric pumping with Symphony® rental breastpumps. **Electric pumping kits sold separately.**

## Symphony® Breastpump Setup


Requires Harmony to Symphony Conversion Kit #67023, sold separately.

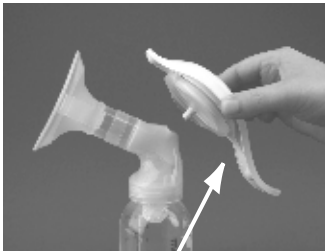


1. Remove handle assembly from Harmony breastpump.



2. Insert hard yellow end of tubing into opening on top of pump.

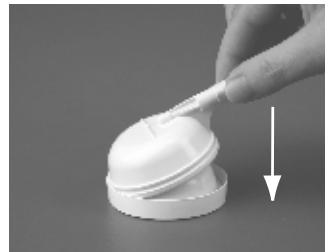

3. Assemble protective membrane and membrane cap.

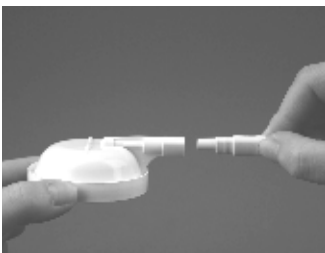

4. Insert the other end of tubing into port on membrane cap.



5. With the palm of your hand, firmly push membrane cap into place on pumping mechanism. Feel for a slight "click" to know it's in place.



Medela, Inc. P.O. Box 660, 1101 Corporate Drive  
McHenry, IL 60051-0660  
Phone: (800) 435-8316 or (815) 363-1166 Fax: (815) 363-1246  
Email: [customer.service@medela.com](mailto:customer.service@medela.com) [www.medela.com](http://www.medela.com)

### Contents may vary from picture.

Manufactured exclusively by and for Medela.

Medela and Symphony are registered trademarks of Medela, Inc.  
Harmony, #1 Choice of Hospitals & Mothers, 2-Phase Expression and SoftFit are trademarks of Medela.  
Patents #4,857,051, 4,929,229. © 2009 Medela, Inc. 1908119 A 0709

### Harmony manual breastpump (NOT FOR RESALE)

The Harmony breastpump is a personal care item that should not be shared between users.

This is a single user product. Use by more than one person may present a health risk and voids product warranty.

This product is warranted by Medela Inc. to the original purchaser to be free from defects in material and workmanship for the period of 90 days.

The Medela Customer Service Team can be reached at (800) 435-8316, Monday-Friday, 7:30 am to 8:00 pm and Saturday, 8:00 am to 12 pm, Central Time.