Bombeo

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
How to use your new Double Pumping Kit with the Medela® Lactina® Breastpump

Article # 6107170W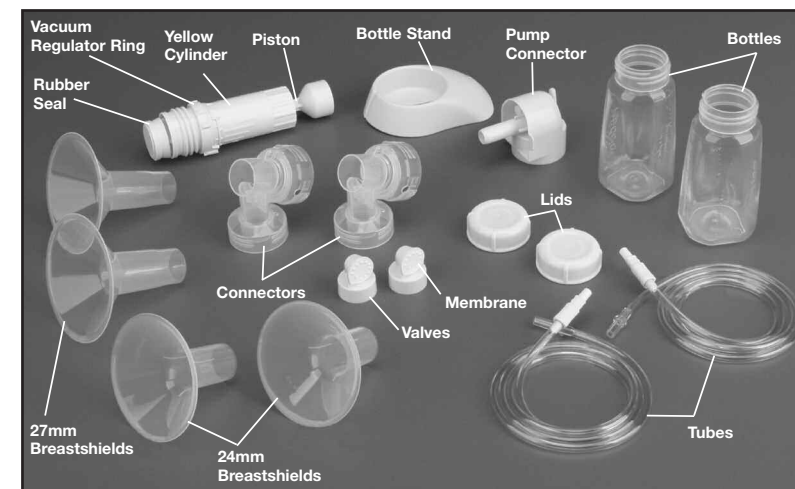

Clean kit by boiling in water before using the first time.

1. Pull off tubes.
2. Take off containers.
3. Unscrew cylinder from cylinder holder.
4. Push piston out of cylinder.
5. Wash in dishwasher or boil.

To boil:





1. Put all parts in large pot.
2. Fill pot with enough water to cover all parts.
3. Boil for 10 minutes.
4. Drain water.
5. Let parts air-dry on a clean surface.
6. When parts are dry, put kit together using directions below as a guide.

Clean in dishwasher or in hot soapy water after each use:

1. Take kit apart.
 2. Take membranes off of valves.
- Important: Do not lose membranes or pump won't work.**
3. Wash parts with hot soapy water.
 4. Only wash tubes and cylinder if milk is in those parts.
 5. Rinse well.
 6. Let parts air-dry on a clean surface.

Kit Assembly

1. Snap membrane onto valve.
2. Push valve into bottom of connector.
3. Push breastshield onto the connector.
4. Screw bottle onto bottom of connector.
5. Push hard, clear end of tubing into opening in back of connector.
6. Push yellow end of tubing into one hole in pump connector.
7. Repeat steps 1-6 to assemble second pump kit for double pumping.

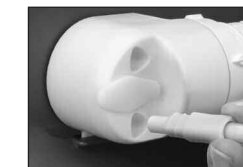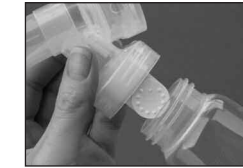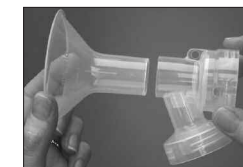
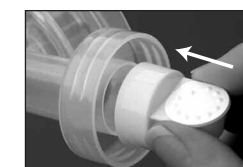
PUMP IS NOW READY TO USE

Piston Assembly

1. Push piston into cylinder, small end first.
2. Twist cylinder into pump connector.
3. Push pump connector into hole on pump.
4. Swing cylinder and piston to the right.
5. Push end of piston into rubber holder on pump. Push all the way down and twist.

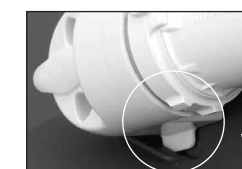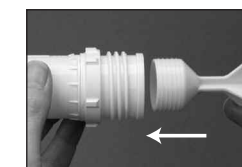Medela, Inc., P.O. Box 660
McHenry, IL 60051-0660, USA
Teléfono: 1-800-435-8316 Fax: 1-815-363-1246
Correo electrónico: customer.service@medela.com

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© 2006 Medela, Inc.




Pumping

- 1 Turn pressure regulator to MIN (minimum).
- 2 Center breastshield over nipple.
- 3 Hold breastshield so entire shield is against breast.
- 4 Turn on pump.
- 5 Turn pressure regulator to comfortable level.
- 6 After a few minutes change to other breast.
- 7 Switch back and forth several times.

For Manual Pumping

- 1 First follow steps 1-4 of KIT ASSEMBLY.
- 2 Slip rubber seal onto piston. The larger side faces the piston handle.
- 3 Push piston into cylinder, small end first.
- 4 Screw cylinder into back of connector.
- 5 Follow pumping directions.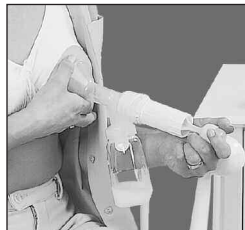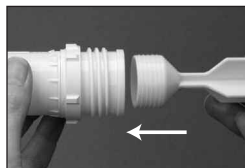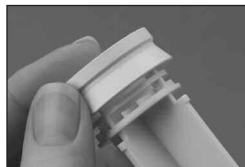

If you have questions call Medela at 1-800-453-8316, or your local WIC office.

Storing and Handling Breastmilk

Storage: (Please check with your hospital for specific storage instructions, if needed.)

1. Wash hands before touching bottles and avoid touching the interior of bottles or caps.
2. Pump into clean bottles.
3. If you do not intend to use the pumped breastmilk within a few days, freeze it.
4. If you are not using the milk immediately, label bottles with the date milk was pumped.
5. Freeze milk in small portions, 2 to 4 ounces per bottle. Smaller amounts thaw more quickly.
6. When freezing milk, do not fill bottles more than 3/4 full to allow room for liquid to expand. Cool milk in refrigerator prior to freezing. Never refreeze breastmilk that has thawed.
7. Use solid lids and seal bottle tightly if you intend to refrigerate breastmilk for several days or freeze it.
8. See chart below for storage guidelines.

Warming/Thawing Breastmilk

1. Place sealed bottle of frozen or refrigerated breastmilk in a bowl of warm water for about 30 minutes. Or hold under warm running water.
2. Do NOT thaw breastmilk in a microwave oven. High temperatures can destroy important nutrients in the breastmilk and there is a danger of scalding baby.
3. Once thawed or warmed, swirl bottle to blend fat, which may have separated during storage. Test for temperature on the inside of your wrist. It should be lukewarm.
4. Feed thawed milk immediately or store in refrigerator for up to 24 hours. Do not let thawed breastmilk sit out at room temperature.
5. If you are taking breastmilk somewhere else, like the baby's caregiver, a cooler with cooling gel packs will help keep breastmilk cold until you reach your destination.

National WIC Breastmilk Storage Guidelines*

	Storage
Refrigerator	*up to 48 hours (Other accepted guidelines range from 2-8 days)
Freezer (inside the refrigerator)	*up to 2 weeks
Freezer (with separate door)	*up to 3 months (Other accepted guidelines range from 3-4 months)

Refrigerated breast milk that is not going to be used within 48 hours should be frozen.

When storing breast milk in the freezer, place containers in the back lower portion of the freezer - where it is coldest - as far away from the door as possible.

Use breast milk with the oldest date first.

* These guidelines vary from source to source. Although somewhat conservative WIC feels these guidelines are appropriate for our participants in the WIC Program.

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