Should I Buy or Borrow a Pre-owned Breastpump?

Many mothers have asked if they can safely sell, purchase, or use a previously owned breastpump. Medela cares about the health and welfare of breastfeeding mothers and babies. Research evidence suggests that certain viruses and bacteria are transmittable through breastmilk. For that reason, Medela follows the FDA policy that states that only hospital-grade pumps (such as the Symphony and Lactina) should be used by more than one person. All other breast pumps should be considered to be single-use devices such as Pump In Style, Swing, and Harmony. For this reason, experts advise against sharing pumps that have been previously owned. The difference between personal use pumps and rental pumps is as follows:

Personal Use Breastpumps

Pump In Style® Advanced
The Metro Bag™

Single Deluxe

Swing™

Personal use pumps that you buy at the store are personal care items, much like a toothbrush. Personal use pumps should never be resold or shared among mothers. The Medela Pump In Style® Advanced has an internal diaphragm that cannot be removed, replaced, or fully sterilized.

Therefore, the risk of cross-contamination associated with re-using a previously owned pump such as the Pump In Style cannot be dismissed, even when using a new kit or tubing. Similarly, the Medela Single Deluxe™ and Swing™

Breastpumps have internal motors that cannot be removed, replaced or fully sterilized. Another consideration when deciding to borrow or even lend a previously owned electric pump is the pump's motor life. A high quality electric double pump might last through the breastfeeding of your second child, or even several children. However, like computers or other electronic products, an electric breastpump has a limited lifetime. Medela guarantees its pump motors with a one year warranty. If you use an electronic pump that has been used for more than one year, there is no guarantee that it will generate as much speed and vacuum as it did earlier in its life. By using your own pump, you can compare the pump's performance with each child. However, if you borrow a pump, you cannot gauge its performance to ensure it is operating at full capacity. Also, multiple women using these single-user products automatically voids the warranty of the Medela product.

Rental Pumps

Rental pumps such as Medela's Symphony and Lactina are designed for multiple users. These pumps have special barriers and filters to prohibit milk from entering the pump motor, which prevent cross-contamination. In addition, each renter uses her own personal tubing and containers to ensure the safe use of these pumps. Medela also provides a selection of breastshields in various sizes to make pumping more comfortable and effective.

Symphony®

Lactina®

Different mothers have different pumping needs and economic means. For this reason, Medela pumps come in a variety of styles and prices. There are also grant programs available for mothers in need.

For information on this and Medela products, ask your local rental station. For product information, availability and pricing, use our Breastfeeding National Network at **1-800-TELL YOU** to find a local rental/retail location. Medela Customer Service is available at **1-800-435-8316** for product questions.

Last Edited: Barbara Wilson-Clay BS, IBCLC
(June 2007)

References:

1 American Academy of Pediatrics: *Report of the Committee on Infectious Disease*, ed 24, Elk Grove Village, Ill, 1997.

Blenkham J: *Infection risks from electrically operated breast pumps*, *J. Hospit Infection* 1989; 13:27-31

Lawrence RA, Lawrence RM: *Breastfeeding: A Guide for the Medical Profession* 6th ed. Elsevier Mosby:Philadelphia,PA. 2005. pg. 781-796.

U.S. Food and Drug Administration (May 2007). *Should I Buy a Used Breast Pump or Share My Breast Pump?* Retrieved June 12, 2007, from U.S. Food and Drug Administration Web site:
<http://www.fda.gov/cdrh/breastpumps/choosing.html#4>

#1 CHOICE OF HOSPITALS AND MOTHERS

Medela, Inc., P.O. Box 660, McHenry, IL 60051-0660 USA
Phone: 1-800-435-8316 or 1-815-363-1166 Fax: 1-815-363-1246 www.medela.com
E-mail: customer.service@medela.com

Single Deluxe, Swing and Double Select are trademarks of Medela.
Medela, Lactina, Symphony and Pump In Style are registered trademarks of Medela.
1547482 D 0607 ©2007 Medela, Inc. All rights reserved.

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Referencias:

1 American Academy of Pediatrics: Report of the Committee on Infectious Disease, ed 24, Elk Grove Village, Ill, 1997.

Blenkharn J: Infection risks from electrically operated breast pumps, J. Hospit Infection 1989; 13:27-31

Lawrence RA, Lawrence RM: Breastfeeding: A Guide for the Medical Profession 6th ed. Elsevier Mosby:Philadelphia,PA. 2005. pg. 781-796.

U.S. Food and Drug Administration (May 2007). Should I Buy a Used Breast Pump or Share My Breast Pump? Retrieved June 12, 2007, from U.S. Food and Drug Administration Web site:

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1547482 D 0607 ©2007 Medela, Inc. Todos los derechos reservados.