How to clean and put together your new Medela® Lactina® Breastpump Kit

Clean kit by boiling in water before using the 1st time.

1. Pull off tubes.
2. Take off containers.
3. Unscrew cylinder from cylinder holder.
4. Push piston out of cylinder.
5. Wash in dishwasher or boil.


To Boil:

1. Put all parts in large pot.
2. Fill pot with enough water to cover all parts.
3. Boil for 10 minutes.
4. Drain water.
5. Let parts air dry on a clean surface.
6. When parts are dry, put kit together using drawing on right as a guide.

Clean in dishwasher or in hot soapy water after each use:

1. Take kit apart.
2. Take membranes off of valves.

Important: Do not lose membranes or pump won't work.
3. Wash parts with hot soapy water.
4. Only wash tubes and cylinder if milk is in those parts.
5. Rinse well.
6. Let parts air dry on a clean surface.

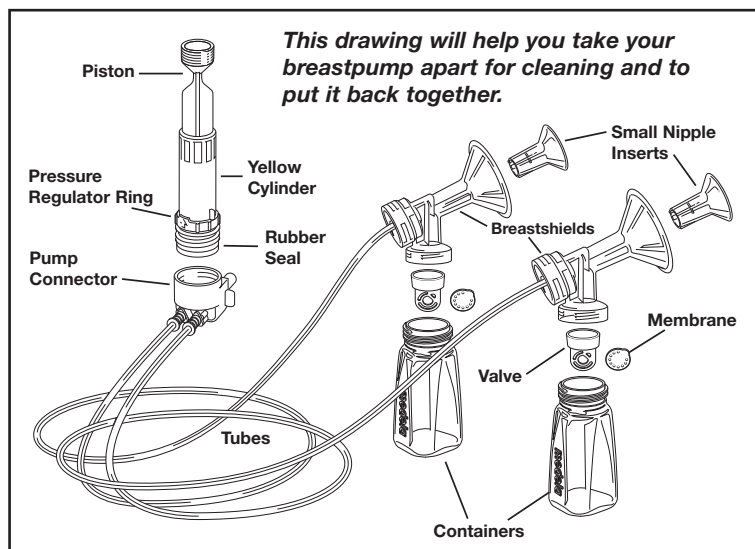For quicker cleaning:

Medela offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection & Anti-Bacterial Wipes for cleaning before and after each pumping session.

CAUTION: This product should only be used with Medela Lactina® breastpumps.

How to use your new Double Pumping Kit with the Medela® Lactina® Breastpump

STEP 1

Wash hands with soap and water.

Important: Kit must be cleaned and put together correctly before use. See top of page.

STEP 2

1 Push piston into cylinder.


2 Screw cylinder into pump connector.

3 Push pump connector into hole on Lactina Breastpump.

4 Swing cylinder/piston to the right.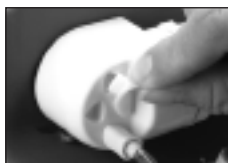

5 If single pumping, put plug into unused tube hole. If double pumping, go to step 6.


6 Push end of piston into white rubber holder.

7 Push all the way down and twist.

STEP 3


1 Put vacuum regulator ring on MIN (minimum).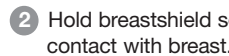

2 Hold breastshield so that entire shield is in contact with breast.

3 Center over nipples and start pumping.

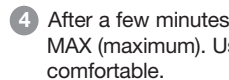4 After a few minutes, try MED (medium) then MAX (maximum). Use the one that feels most comfortable.

Now your pump is ready to use.

If you have questions, call Medela at 1-800-435-8316 or your local WIC office.

Breastmilk Handling & Storage

- Wash hands with soap and water.
- Always pump and save breastmilk in a clean container, such as a bottle.
- Write date on container.

Guidelines for storing breastmilk

	Breastmilk Storage (For Healthy Term Babies)				
	Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Self-contained Refrigerator Freezer unit	Deep Freezer
Freshly expressed breastmilk	4 hours at 66-72°F (19-22°C)	24 hours at 59°F (15°C)	5-7 days at 32-39°F (0°C)	3-4 months	6-12 months at 0°F (-19°C)
Thawed breastmilk (previously frozen)	Do not store	Do not store	24 hours	Never refreeze thawed milk	Never refreeze thawed milk

Thawing & warming of breastmilk:

- Put container in warm water.
- Gently swirl container and check temperature of breastmilk.
- NEVER microwave breastmilk.
- NEVER refreeze breastmilk.

¹ Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: Pediatrics, vol. 97, No. 4, April 1996, pp 492-497. (4 hours at 77° F/25° C).

² Sosa, Roberto; Barness, Lewis: AJDC, Vol. 141, Jan. 1987.

³ Lawrence R, and Lawrence R: Breastfeeding: A Guide For the Medical Profession, 1999, p.894.

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